


New England Low Vision *and Blindness*

Bringing Hope Through Technology, Training & Care


info@NELowVision.com • www.NELowVISION.COM

Battery Installation Instructions

1:

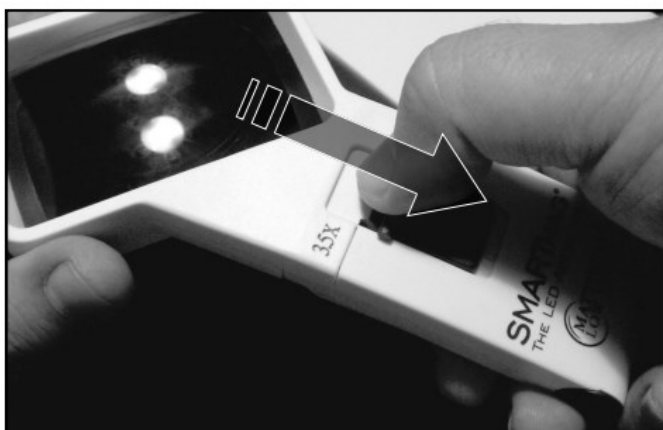

Place your finger at the top end of the switch and pull down to unlock the battery area.

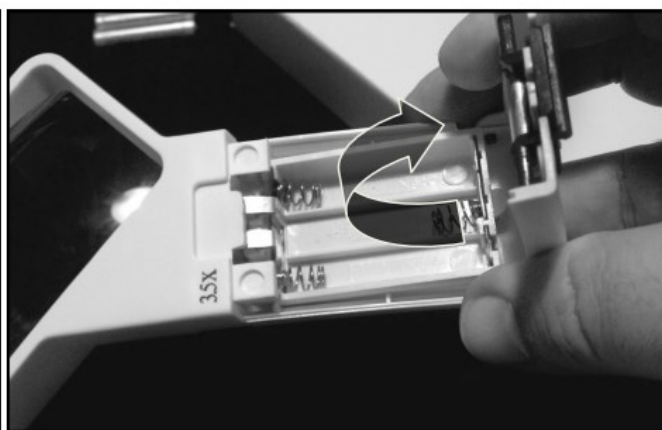

Grip the sides of the battery cover and lift up to reveal the battery compartment.

2:



The battery compartment holds 3 "AAA" alkaline batteries.



Make note of the positive and negative sides of the battery.

3:




Slide the battery into the slot with the negative (-) end pushing the spring. Be careful to avoid pushing down on the silver metal tabs (circled).



The positive ends (+) of the “AAA” batteries should be oriented as shown above.

Roll the batteries with your finger to make a good contact.

4:

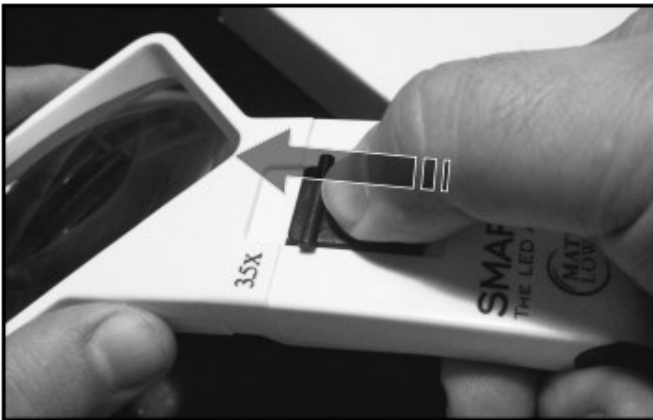

Close the cover and slide the switch one (1) click toward the lens to secure it.



Slide the switch forward once more to illuminate the LED.

If your magnifier does not illuminate, please try two procedures:

1. Double check that the batteries are secure in their slots, try to press down and roll them with your finger.
1. The two silver metal prongs may have been bent down when inserting batteries, causing loss of contact. If so, gently bending the prongs up slightly should correct this. If the prongs are bent too far, the switch will not move to the on position—if this happens gently bend the prongs down a little.

**MR.
MAGNIFIER**

a product line of New England Low Vision and Blindness
info@NELowVision.com | www.NELowVision.com