

MagniLink S ChromeViewer 4.0 Hotkey Guide

General Commands – work on all tabs

Shortcut Command	Function
Alt + 1	Go to Camera Tab
Alt + 2	Go to OCR Tab
Alt + 3	Go to Notepad Tab
V	Switch Between Tabs
1	Natural Colors
2	Positive Colors Forward
Shift + 2	Positive Colors Backwards
3	Negative Colors Forwards

Shift + 3	Negative Colors Backwards
4	Start OCR
5	Add Note
Ctrl + P	Take / Save a Picture
Ctrl + M	Start / Stop Recording
8	Split Screen Mode
9	Info / Settings Window
L	Connect LVI Camera
Up Arrow	Increase Magnification
Down Arrow	Decrease Magnification
Right Arrow	Increase Contrast
Left Arrow	Decrease Contrast

Ctrl + Right Arrow	Ref. Line Right / Down
Ctrl + Left Arrow	Ref. Line Left / Up
Ctrl + R	Rotate Image Clockwise
Ctrl + Shift + R	Rotate Image Counterclockwise
Ctrl + I	Mirror Image
Ctrl + F	Freeze Image
Ctrl + 6	Reduce Lamp Brightness
Ctrl + 7	Increase Lamp Brightness
A	Pan Left
D	Pan Right
W	Pan Up
S	Pan Down

J	Auto Focus On/Off
Shift + Up Arrow	Reduce Focus Distance
Shift + Down Arrow	Increase Focus Distance
Ctrl + H	Toggle Hide or Show Menu
Ctrl + Plus	Increase Menu Size
Ctrl + Minus	Decrease Menu Size
Ctrl + L	Toggle Whole & Cropped Image

OCR Commands

Shortcut Command	Function
1	Pause / Play Reading
2	Stop Reading
3	Previous Word / Sentence / Letter
4	Next Word / Sentence / Letter
5	Switch Navigation Between Options
B	Change Display Mode
7	Reduce Speaking Speed
8	Increase Speaking Speed

Notepad Commands

Shortcut Command	Function
Ctrl + Z	Undo
Ctrl + Shift + Z	Redo
Ctrl + S	Save
Delete	Delete Note
Ctrl + R	Rotate Image Right
Ctrl + Shift + R	Rotate Image Left
1	Select Ballpoint Pen
2	Select Highlighter
3	Select Text Tool
4	Select Magnification & Panning Tool

	5	Select Eraser
Ctrl + 1		Previous Color of Drawing Tool
Ctrl + 2		Next Color of Drawing Tool
Ctrl + 3		Reduce Size of Drawing Tool
Ctrl + 4		Increase Size of Drawing Tool

Please contact LVI America @ 888-781-7811 or info@lviamerica.com.

For more information about the MagniLink S, go to our webpage [MagniLink S Product Information](#).