

Software information MagniLink ChromeViewer

Last Updated: September 25, 2024

Software information MagniLink ChromeViewer

Due to a change from Google in their Chrome OS it is no longer possible to connect to an LVI camera through the "connect button" in MagniLink ChromeViewer if the application has been started from the Chrome OS "Extensions". The manufacturer has reported this to Google and hopes they make a change in future versions.


Instead, the application should be started from "App Launcher" or attached to the Start Area and started from there. Then the "connect button" works as it should. How this is done is described below and the same information is also available in the User Manual.

If you want access to MagniLink ChromeViewer in the App Launcher with the option to attach the application to the start area, do as follows:


1. Launch MagniLink ChromeViewer.
2. Go to Settings.
3. Press the **Copy URL** button located under the About setting page to copy the address to the app.
4. Launch the Google Chrome browser.
5. Select the address field and paste the copied address either by pressing **Ctrl + V** or by right-clicking on the address field and selecting Paste. Press Enter.
6. Under **More** at the top right, press **More Tools**, and then **Create Shortcut**.

Software information MagniLink ChromeViewer

Last Updated: September 25, 2024

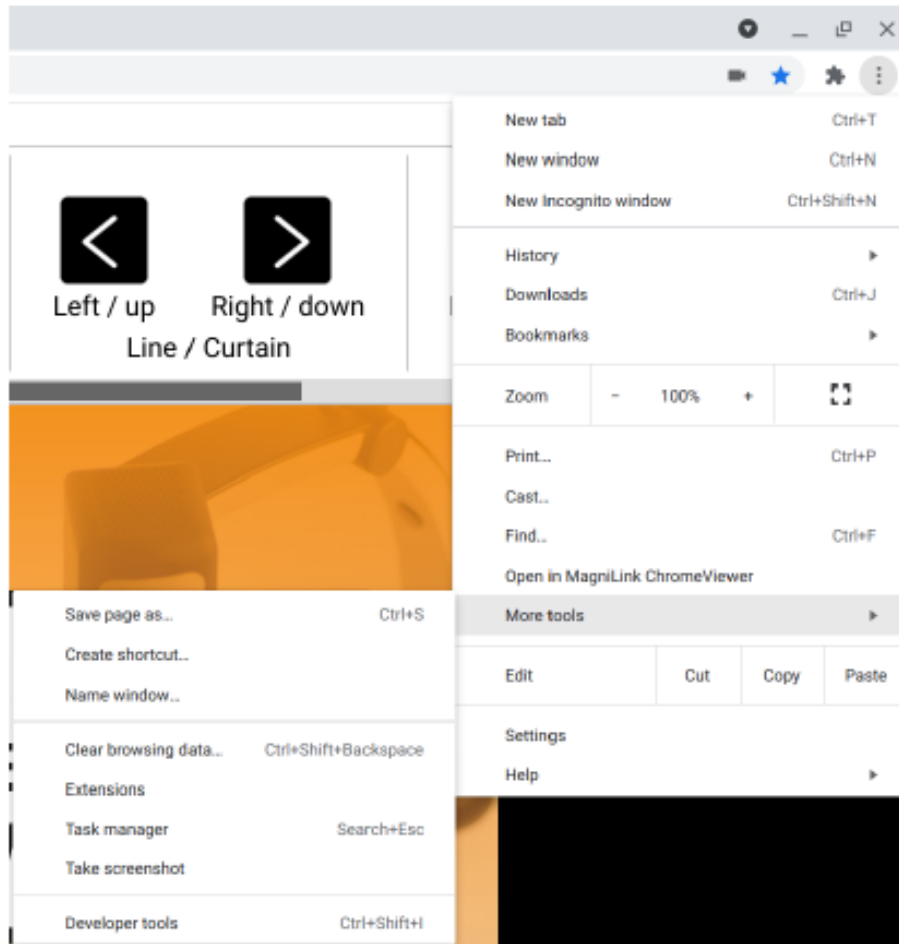

7. Select Open as window, then press Create.

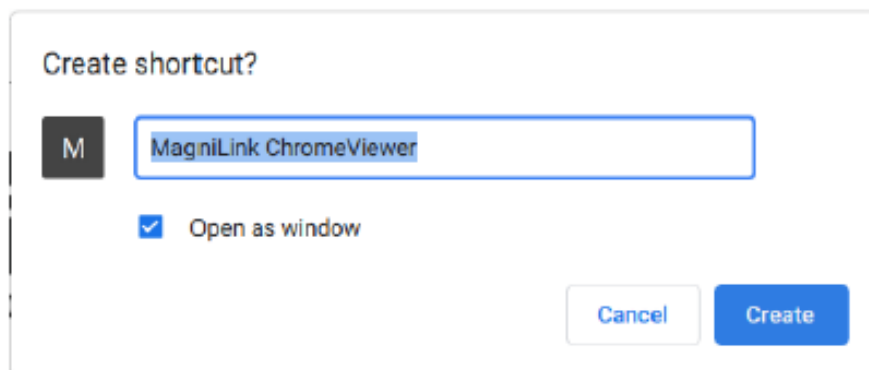

In App Launcher you can also attach MagniLink ChromeViewer to the start area.

Software information MagniLink ChromeViewer

Last Updated: September 25, 2024

Disclaimer:

The content and images in this document have been provided by the manufacturer. While New England Low Vision and Blindness has made every effort to ensure the information is accessible and designed with the visually impaired in mind, we do not claim responsibility for the content's accuracy, completeness, or timeliness. For specific questions or concerns, please consult the manufacturer directly.