

## LyriQ AI Software Update Instructions

Last Updated: April 24, 2026

---

### LyriQ AI Assistive Text-to-Speech Reader Mandatory Software Update Notice

**Important:** A critical software update is required for all LyriQ AI devices to continue functioning properly.

New England Low Vision and Blindness has been notified by the manufacturer that all current LyriQ AI users must complete this update to maintain uninterrupted access to AI-powered text-to-speech functionality.

---

### What You Need to Know

- This update is **mandatory** for continued device performance
  - The update is completed over WiFi
  - This update installs **AI service version 2.08**
  - **No features or functionality will change** as a result of this update
- 

### Before You Begin

Ensure your LyriQ AI device is:

- Powered on
  - Connected to a stable WiFi network
- 

### Step-by-Step Update Instructions

1. Enter the **Main Menu** by pressing:
    - **BACK + FASTER** buttons
  2. Navigate to **About**:
    - Use the **SLOW** button to scroll
    - Press **PAUSE** to select
  3. Select **Updates**:
    - Scroll to "Updates"
    - Press **PAUSE** to enter
  4. Check for updates:
    - Scroll to "**Look for Updates**"
    - Press **PAUSE** to select
  5. Install version 2.08:
    - The device will locate update version **2.08**
    - Press **PAUSE** to begin installation
    - The update will install automatically
- 

### After the Update

Once complete, your LyriQ AI will continue operating as normal with improved system stability. No changes to user experience, features, or functionality will occur.

---

## **LyriQ AI Software Update Instructions**

**Last Updated: April 24, 2026**

---

### **Need Assistance?**

If you need help connecting to WiFi or completing this update, contact New England Low Vision and Blindness for support.

We are here to ensure your assistive technology continues to perform reliably and effectively.

---

### **New England Low Vision and Blindness**

Website: <https://NELowVision.com>

Email: [Info@NELowVision.com](mailto:Info@NELowVision.com)